# FAB Cobalt-Chrome T4

|                                                                 |    |
|-----------------------------------------------------------------|----|
| DE - Gebrauchsanweisung FAB Cobalt-Chrome T4 .....              | 2  |
| EN - Instruction for use FAB Cobalt-Chrome T4.....              | 3  |
| FR - Mode d'emploi FAB Cobalt-Chrome T4 .....                   | 3  |
| CZ - Návod k použití FAB Cobalt-Chrome T4.....                  | 4  |
| EL - Οδηγίες χρήσης FAB Cobalt-Chrome T4 .....                  | 5  |
| ES - Instrucciones de uso de FAB Cobalt-Chrome T4 .....         | 6  |
| HR - Upute za uporabu proizvoda FAB Cobalt-Chrome T4 .....      | 7  |
| HU - Használati útmutató a FAB Cobalt-Chrome T4 termékhez ..... | 8  |
| IT - Istruzioni per l'uso di FAB Cobalt-Chrome T4 .....         | 9  |
| PL - Instrukcja stosowania stopu FAB Cobalt-Chrome T4 .....     | 10 |
| PT - Instruções de utilização do FAB Cobalt-Chrome T4.....      | 11 |
| RO - Instrucțiune de utilizare FAB Cobalt-Chrome T4.....        | 12 |
| SE - Bruksanvisning för FAB Cobalt-Chrome T4 .....              | 13 |
| DA – Brugsanvisning for FAB Cobalt-Chrome T4.....               | 14 |
| ET – Kasutusjuhend FAB Cobalt-Chrome T4 .....                   | 15 |
| LT – Naudojimo instrukcija FAB Cobalt-Chrome T4 .....           | 16 |
| LV – Lietošanas pamācība FAB Cobalt-Chrome T4.....              | 17 |
| NL – Gebruiksaanwijzing FAB Cobalt-Chrome T4.....               | 18 |
| SK – Návod na použitie zliatiny FAB Cobalt-Chrome T4.....       | 19 |
| SL – Navodila za uporabo FAB Cobalt-Chrome T4 .....             | 20 |
| TR -FAB Cobalt-Chrome T4 Kullanım Talimatı.....                 | 21 |

**MINDFAB**

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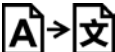
Stand / Status 12/2025 as  
Seite 2 von 21

# EN - Instruction for use FAB Cobalt-Chrome T4

**Intended use**  
**FAB Cobalt-Chrome T4** is a medical device for the milling of crowns and bridges.

**Only for professional user (Dental Technician, Dentist).**

**Indication**  
Crowns and bridges for metal-ceramics up to 16 units (max. 4 units in the anterior region and max. 3 units in the posterior region), superstructures, bars.

**Contraindication and Patient group**  
In case of known allergic reactions to any of the ingredients.

The product is intended exclusively for use in adult patients. The use in vulnerable groups of people - in particular children, pregnant women and breastfeeding women - is contraindicated, as an increased sensitivity to potentially released metal ions cannot be ruled out in these groups.

**Milling**  
**FAB Cobalt-Chrome T4** is constructed for CNC milling machines. Please follow the instructions and parameters of the respective manufacturer of CAM Software and the CNC milling machine.

**CAD**  
The design should be done with appropriate CAD software. Please consider an anatomically reduced framework design for the veneering with ceramic. The wall thickness should not be less than 0.3 mm. Choose a sufficient connector dimension (6 - 9 mm<sup>2</sup>). Sharp edges and undercuts should be avoided.

**Cutting out the frameworks from the Blank**  
Remove the milled frameworks with suitable cutting tools and smoothing the supports.

**Soldering / Laser welding**  
**FAB Cobalt-Chrome T4** can be soldered with all suitable solder. **FAB Cobalt-Chrome T4** parts should not be soldered with gold or palladium solder. **FAB Cobalt-Chrome T4** is also ideally suitable for laser-welding.

**Preparation before ceramic veneering**  
The frameworks can be elaborated with standard carbide cutters, look for smooth transitions and avoid overlapping material. Please use the same cutter for one alloy to avoid contamination. The minimum thickness of the prepared coping should not be less than 0.3 mm. It's recommended to sandblast the frames with minimum 110 µm of Aluminium oxide with 3-4 bar and clean with steam cleaner. Oxide firing is not mandatory but can be done as an option for 5 minutes at 980°C (cleaning firing). The frame needs to be sandblasted again to remove the present oxide layer thoroughly. In the end the cleaning by steam cleaner is mandatory. If you use a ceramic bonder, please consider the instruction for use of the manufacturer.

**Handling conditions / Safety**  
**Metal dust is harmful to health. Use when grinding and sandblasting dust extraction and respirator with filter FFP3 – DIN EN 149.**

**Residual risks and side effects**  
If the instructions are observed during the production processes, incompatibilities with CoCr alloys are extremely rare. In case of a proven allergy against an ingredient of this alloy, the alloy must not be used for safety reasons. In exceptional cases, electrochemically induced, local irritations have been reported. When different alloy groups are used, galvanic effects might occur. Please inform your dentist regarding the residual risks and side effects. Any serious incident that involves the product must be reported to the manufacturer and the competent authority in the accorded country. The SSCP is available at <https://ec.europa.eu/tools/eudamed> and [www.adentatec.com](http://www.adentatec.com).

**Disinfection of the dental prosthesis before insertion**  
Workpieces from the dental laboratory must be subjected to immersion or spray disinfection before insertion into the patient's oral cavity and then rinsed under running water.

**Single-use**  
Used milling disc should be not processed for further use as a medical device.

**Disposal Instructions**  
Please dispose of metal residues and dust in an environmentally friendly manner. Do not allow waste to enter groundwater, water or sewage systems. Contact waste exchanges for recycling. Outer packaging can be disposed of in paper waste.

**Storage conditions**  
Temperature, humidity or light has no effect on the product properties

*Our information and recommendation are based on the state of the art in science and technology and has to be considered correct to the best of our knowledge and experience on this day. The above version shall replace any previous versions.*

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Adentatec GmbH
MINDFAB GmbH